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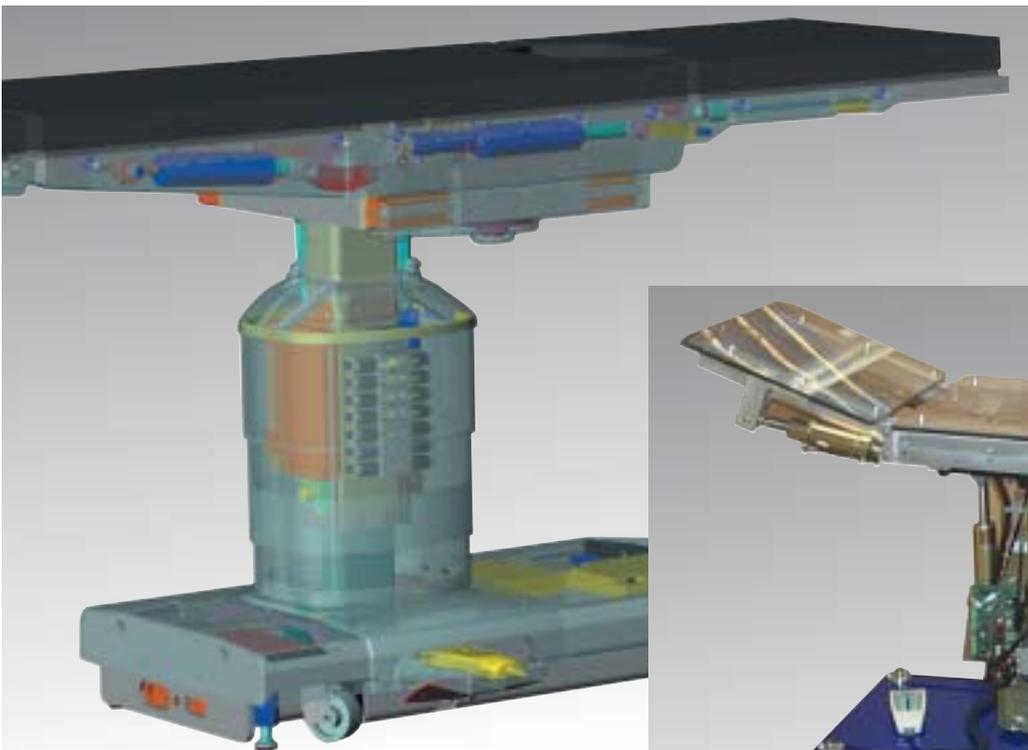
Standard Lift Columns C6 for Operating Tables, ready to fit

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A modern and growing company introduces itself:

With more than 25 years of experience in micro hydraulics we have acquired not only national but also interational fame as „the specialist“ in this field.

Due to continuous improvement and development for patient positioning (such as operating tables) we have obtained a position as market leader.

In a close co-operation with HOERBIGER Automatisierungstechnik, HOERBIGER Micro Fluid is a competent and effiecient partner for the industry in niche markets.

We are audited according to:
DIN EN ISO 9001: 2000
EN ISO 13485: 2000
VDA 6, Teil 1

We understand ourself as partner for the client, in order to realize together successful solutions.

HOERBIGER Micro Fluid in Barbing has 50 employees and does see itself as partner and pioneer for the industry for system solutions in the field of automation, automotive engineering and patient positioning.HOERBIGER worldwide has more than 4300 employees.

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Berchtold
Bicakcilar
Brumaba
Famed
Maquet
Medifa Hesse
Mediland
Merivaara
Schmitz
Siemens
Storz medical

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Reference List

USA
Turkey
Germany
Poland
Germany
Germany
Taiwan
Finland
Germany
Germany
Suisse

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General information about patient positioning and its weight distribution

Torque at the Trendelenburg axis is caused by the patient, the top plate and the surgery equipment.

To consider the max. patient weight possible, a normal weight distribution of the patient, the top plate and the accessories are assumed.


Additional suppositions:

- no slide movement of the table top
- Trendelenburg cylinder at the head section
- patient head is situated at the end of the head section
- patient weight 255 kg, distribution according to DIN EN 60601

	Hebelarme der Schwerpunkte <i>Lever arms of the centre of mass</i>	Gewichtskräfte der Komponenten <i>Weight of the components</i>	Drehmoment um Trendelenburgachse <i>Torque at the Trendelenburg axis</i>
	mm	kg	Nm
Patient / <i>Patient</i>	245	255	625
Lagerfläche / <i>Positioning surface</i>	280	80	224
OP-Zubehör / <i>Surgery accessories</i>	800	20	160
OP-Zubehör / <i>Surgery accessories</i>	500	20	100
Summe / <i>Total</i>		375	1109

Max. Patientengewicht, Kräfte, Momente

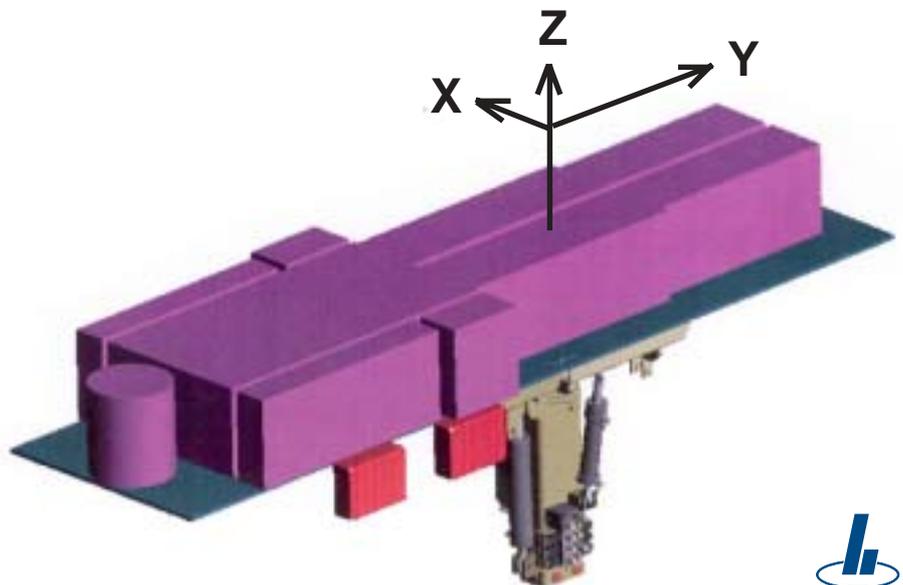Max. Patient weight, Forces and Torques

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Lift Column C6

From our product range of various types of lift columns we present to you the different versions of lift column C6.

The basic version consists of 3 basic movements (stroke, lateral, tilt), which can be operated either manually (description page 6) or electrically (description page 8). Three extra functions (leg, back, foot lock) can be realized in addition (description page 10).

The power supply has to be provided by the manufacturer of the operating table.

All versions are closed systems whose function and tightness are tested and ready to be installed in your operating table.

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Movements	
stroke	360 mm
lateral	$\pm 30^\circ$
tilt	$\pm 20^\circ$
Speed (without load!)	approx. 9 mm/s

Forces (theoretically)	
stroke	5000 N
Torque nominal*	
x-axis	1100 Nm
y-axis	250 Nm
z-axis	500 Nm

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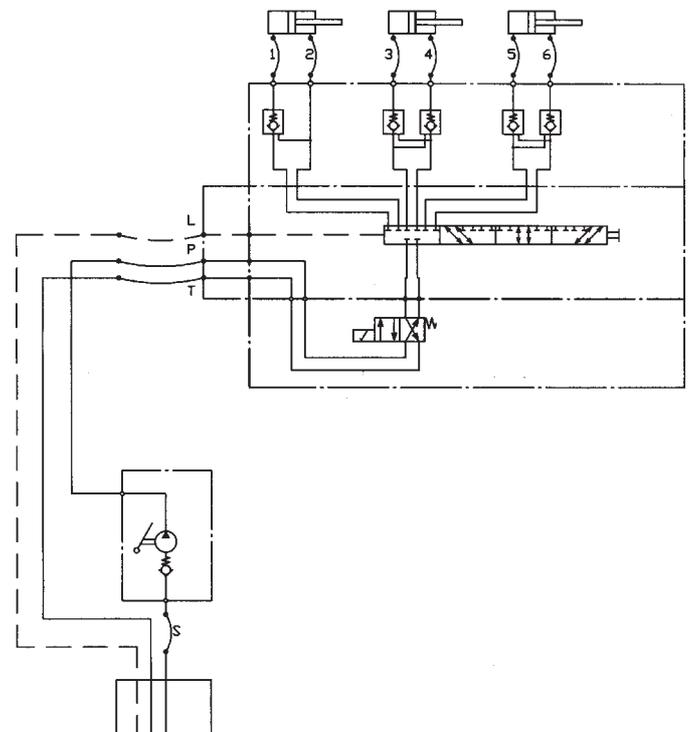
Lift Column C6 manual, 3 movements

The movements can be activated by operating the foot pump (e.g. by pedal). The position of the rotary valve (5) has to be put into the desired position. The frame (2) moves. A change of the movement direction is achieved by the reversing valve (6).

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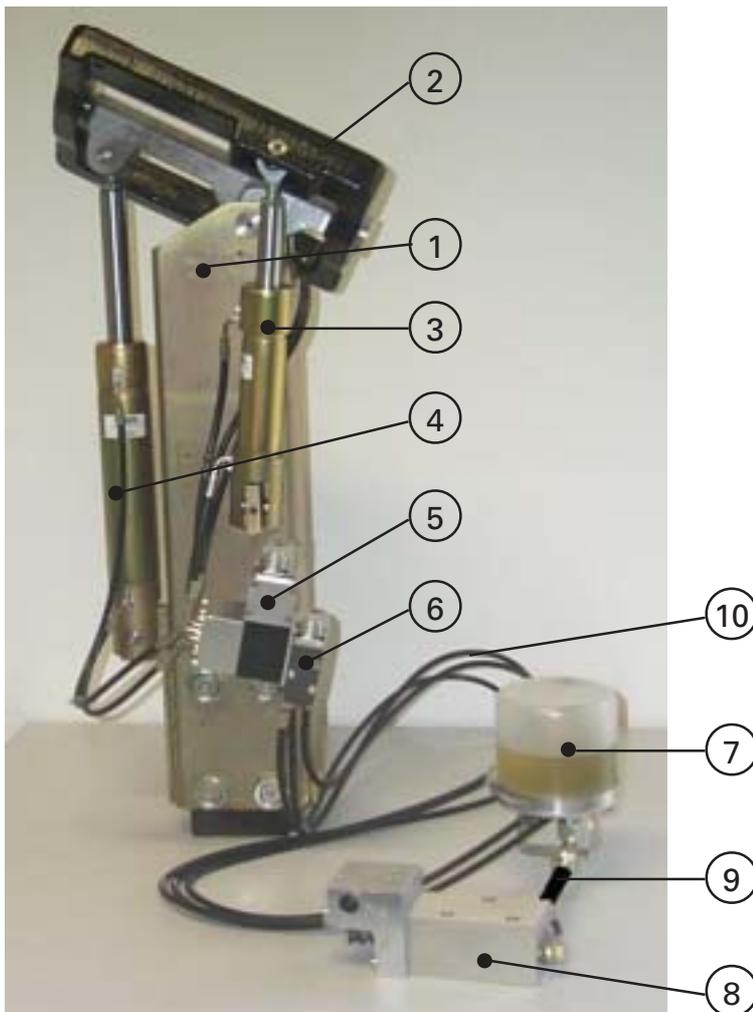
manual, 3 movements

Delivery volume:

- (1) Linear unit complete
- (2) Outerframe
- (3) Cylinder tilt
- (4) Cylinder lateral
- (5) Rotary valve
- (6) Reversing valve
- (7) Tank
- (8) Foot pump
- (9) Suction hose
- (10) Hoses

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Lift Column C6 electric, 3 movements

The power supply 24 VDC is effected externally by an akku (not included in the delivery volume).

After the akku is connectrd the desired movements can by activated by pressing the buttons on the hand panel.

Particularities:

- if no button is pressed on the hand panel the electric system is switched off after 30 s.
- Two LED displays (Ready & Service) indicate the operating states

Option:


- horizontal zero position

Hydraulic

Power unit:	
Nominal voltage	24 VDC
Power consumption	max. 15 A
Protection class	IP54
Working pressure	140 bar

Electronic

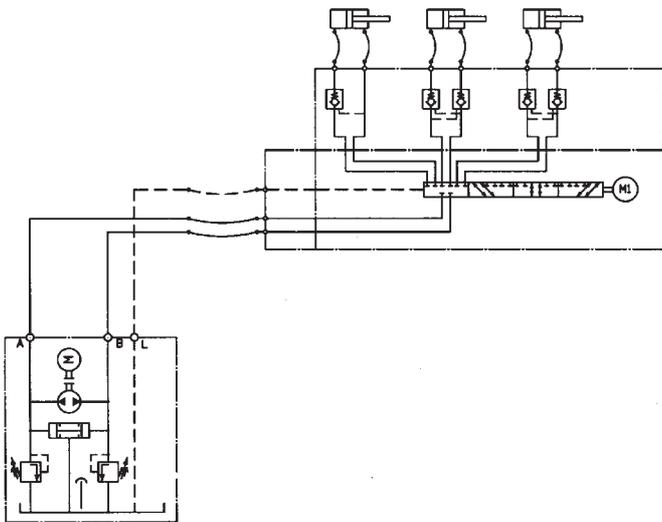
EMC tested according to EN60601-1-2:1993 (case of the lift column in stainless steel 2 mm)

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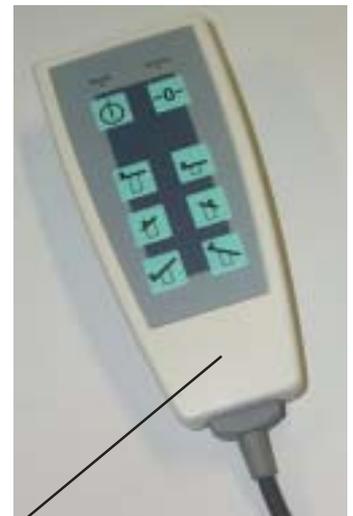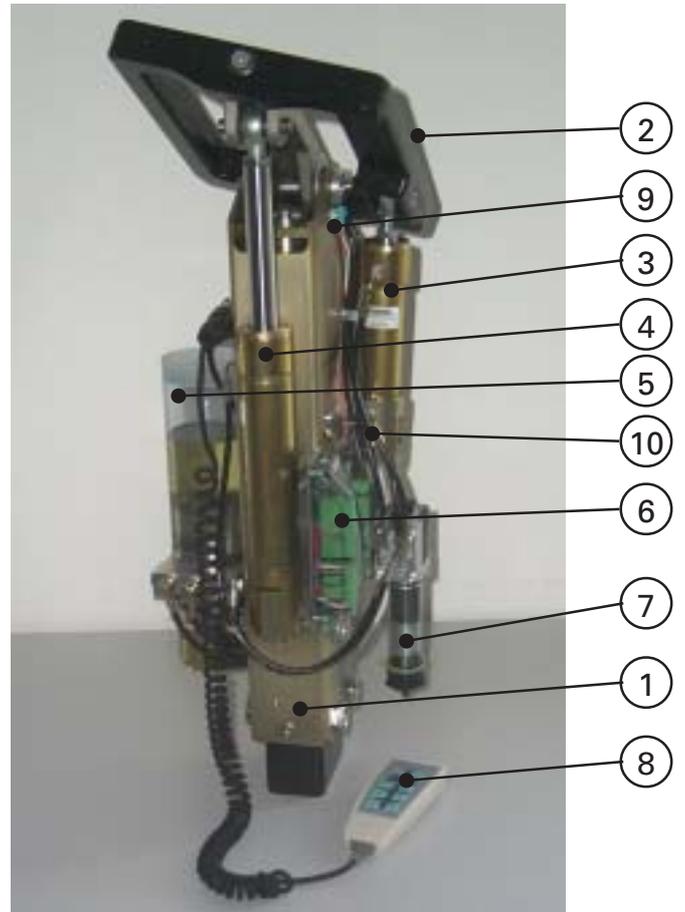
electric, 3 movements

Delivery volume

- (1) Linear unit complete.
- (2) outer frame
- (3) Cylinder tilt
- (4) Cylinder lateral
- (5) Power unit A 08
- (6) Electronic board complete. (MPC)
- (7) rotary valve complete
- (8) hand panel incl. spiral cable
- (9) cable harness
- (10) hoses

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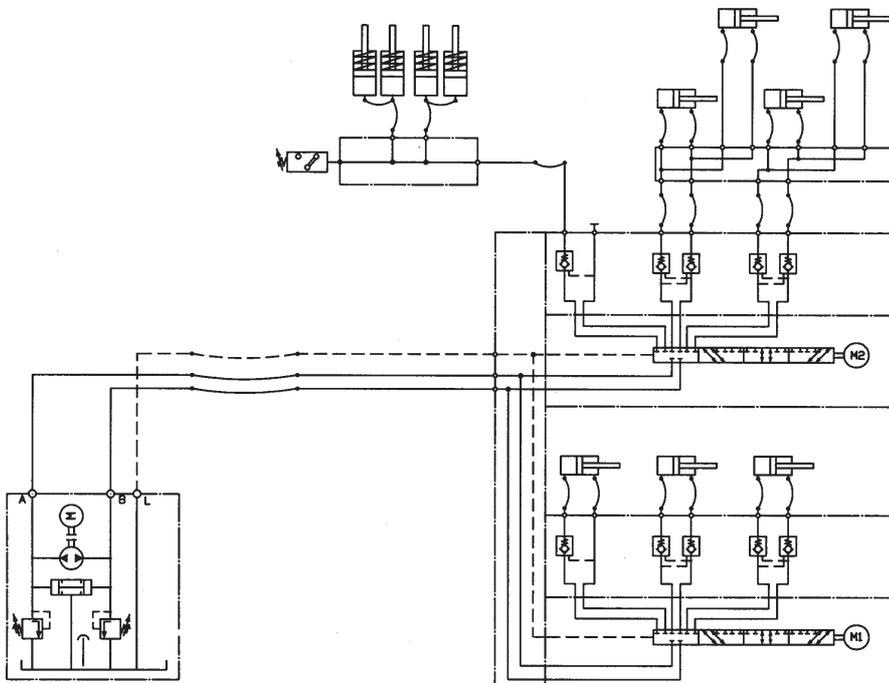
Lift Column C6 electric, 6 movements

Basic movements see lift column C6 with 3 movements (page 8).


Press corresponding button to activate the additional functions leg and back plate.

The three basic movements (see lift column C6 – page 8) and the additional functions back and leg plates can only be activated after pressing the button table unlock (No.7)

Particularity:
The foot locking is secured by a pressure switch.

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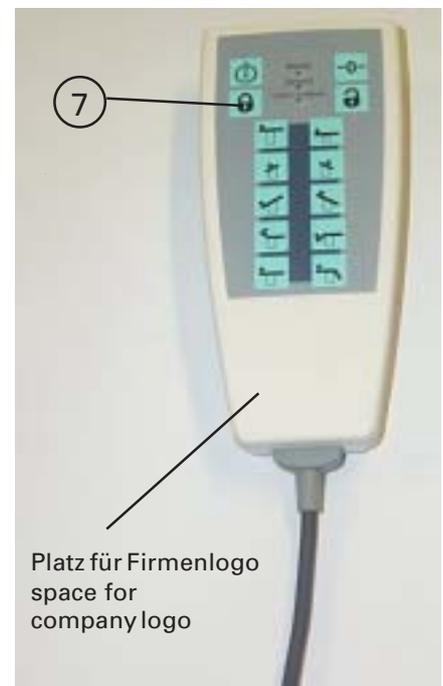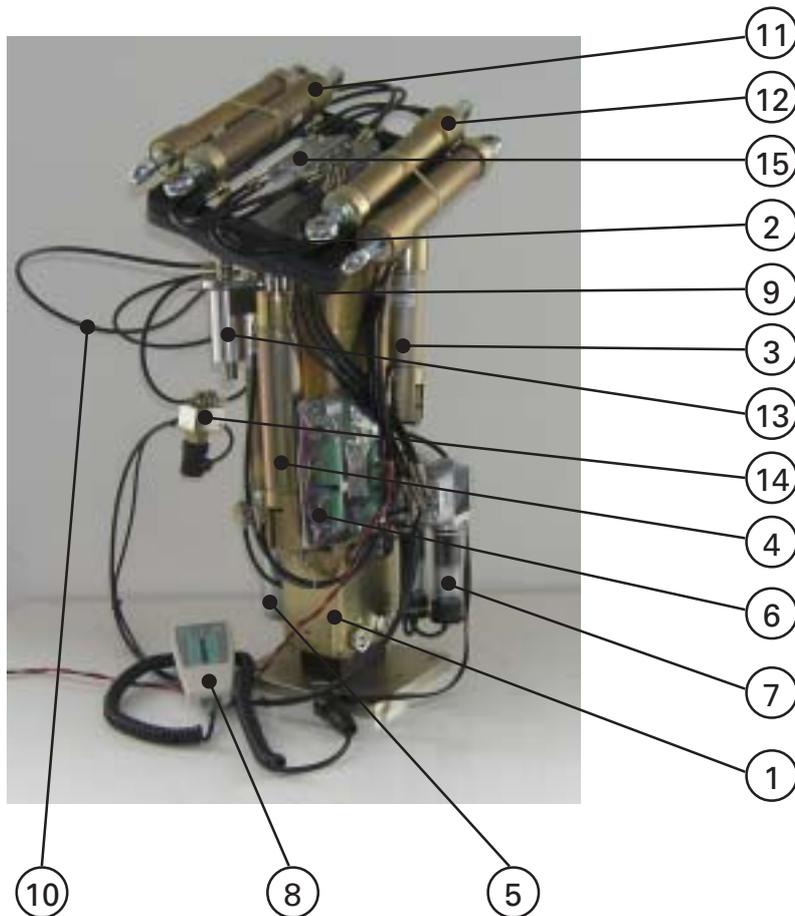
electric, 6 movements

Delivery volume

- (1) Linear unit complete
- (2) outer frame
- (3) Cylinder tilt
- (4) Cylinder lateral
- (5) Power unit A 08
- (6) Electronic board complete (MPC)
- (7) 2 rotary valves complete
- (8) hand panel incl. spiral cable
- (9) cable harness
- (10) Hoses
- (11) 2 cylinders to move the leg plate incl. hoses (leg cylinders)
- (12) 2 cylinders to move the back plate incl. hoses (back cylinders)
- (13) 4 cylinder to lock the table incl. hoses
- (14) Pressure switch
- (15) Distributor block

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Cylinder LV (double-acting)
version pin connector and
version banjo bolt
and
Cylinder LF (single-acting with
spring return)

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Rotary valve with pilot-operated
checkvalve

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The statements made here are typical values, they are not guaranteed characteristics in the legal sense.
Technical modifications may be undertaken. We do not accept liability for any errors.

HOERBIGER Micro Fluid GmbH · Borsigstraße 11 · 93092 Barbing/ DEUTSCHLAND
Tel. +49 (0)9401 785 - 0 · Fax 785 - 50 · E-mail info-hmf@hoerbiger.com

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